

Eye-Pal SOLO



Eye-Pal Solo

- ▶ Auto-Reading talking appliance
- ▶ Reads books, magazines and other printed material
- ▶ Increase or decrease reading speed
- ▶ Volume control
- ▶ Pause/Resume button
- ▶ Optional hand motion activated
- ▶ No computer connection required

Eye-Pal Solo LV

- ▶ All Eye-Pal Solo features, plus...
- ▶ Magnify snapshot page on monitor
- ▶ Highlighted text synced with speech
- ▶ Save up to 4 pages of text
- ▶ Column reading mode
- ▶ Continuous line scrolling
- ▶ Switch between female or male voice
- ▶ Read English or Spanish text
- ▶ Live view with variable magnification



Freedom Vision
Providing the freedom to read again

800-961-1334

www.freedomvision.com

Eye-Pal SOLO...just place your document, then listen

Eye Pal SOLO will virtually read to you on its own ... no buttons to press, no menus to set up, no prior instructions to read, no training required, and no PC connection required. Eye-Pal SOLO is easy to use out of the box. Set up takes just a few minutes. Just insert the camera stand, connect to the camera's USB port, insert the power cord into a wall outlet, press the Power button to power On, place the page of a magazine, newspaper, book, or a piece of mail under the camera, then within seconds Eye-Pal SOLO will begin reading aloud.


Eye-Pal Solo will read the full page, a sentence or word. All controls are located on the front panel. Set the speech volume with the Volume dial, and increase or decrease the speech rate with the Increase or Decrease buttons. Pause or Resume simply by pressing the white button on the front panel. Go back one sentence at a time by pressing the red button. Or, activate hand motions, then with a smooth waive of your hand under the camera you can also instruct Eye-Pal Solo to pause, resume or go back to the previous sentence. To read a new page, just remove the current document, then position the new page under the camera. It's that simple.

Eye-Pal Solo LV offers auto-reading in speech mode, as well as adding two magnification display modes. Connect to a VGA monitor to view the snapshot of the printed page, then select the magnification level, the image display text/background colors, and display view (full page reading, column reading, or continuous line reading). Read the page with synchronized speech and magnification with the spoken word highlighted, or turn off the speech and read the magnified display at your own pace. Or, switch to live camera mode and magnify up to 12x to read your mail, view photographs, handwritten letters, a prescription bottle or grocery item, sign a check or fill in a form. This live video magnification camera feature is a supplemental capability, however, it is not a substitute for a CCTV video magnifier.

Eye-Pal Solo LV can switch from a female to a male voice, recognize English or Spanish text (more languages coming soon), describe page layout, display in a Continuous Line scroll or in a Column scroll mode. These valuable features provide added flexibility for easy access to a wide variety of print, handwriting, photographs, bottles, cans, and more.



Eye-Pal SOLO LV keypad

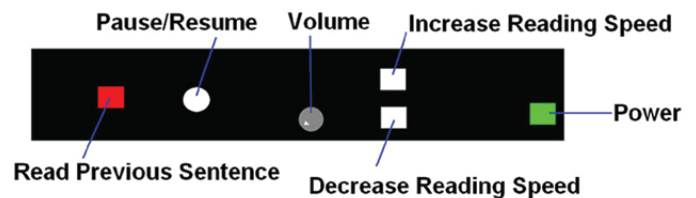

Front panel control